

# Stem Cell Gel

## Contains:

Aloe Vera Gel	
Nano Silver	32 ppm / 100 ppm
Glycerine	15% w/w

## Indications:

Stem Cell Gel is used for the management of wounds including Pressure Ulcers, Venous Ulcers, Dermal lesions, Second degree burns, Donor sites.

## Directions for Use:

- 1) Remove all dressing / ointment / previous bandages if any;
- 2) Bathe / rinse the wound with soap and water;
- 3) Apply a thin coat of gel on the surface of the wound / area of concern;
- 4) Cover it with a gauze / dressing and gently soak the gauze with nano silver spray until moist.
- 5) Now apply a dry piece of gauze to cover the wound with bandage or adhesive tape (plaster).
- 6) Repeat only spraying the wound with nano silver two more times a day at 6 to 8 hour intervals;
- 7) Repeat above procedure daily till satisfactory results are achieved.
- 8) Do not use saline, cetavlon, betadine, hydrogen peroxide or any other agent normally used in conventional wound toilet. Strictly adhere to the instructions for best results

## Contraindications:

There are no absolute contraindications but its utility is limited when used heavily exudating wounds or wound covered with thick eschar.

## Storage:

Store under Refrigeration (4° C. to 8.0° C. / 38° F. to 48° F.) to maintain full potency and extend shelf life.

## Note:

Local wound care cannot overcome the deficits of unrelieved malnutrition, pressure compromised blood flow. Interventions must be instituted to reduce / relieve these factors that impair wound healing in order to achieve optimum results.

## References: (Case Studies)

<http://www.space-age.com/NanoSilverStemCellActivation/pdf>

<http://www.space-age.com/FingertipRegeneration.pdf>